

CAM Device Removal



- When the monitor has completed the recording period (2 day or 7 day based on the unit), have the owners return to your hospital for removal or provide these instructions so they can accomplish this at home.
 - We find it usually requires two people to remove it, so returning to your hospital may be the best idea.
- Inside the diary there is a packet of Adhesive Remover (cloth patch in a small tear pouch). We will generally supply you with extra packets of adhesive remover as well. If necessary, isopropyl alcohol can also be sprayed/dripped into the adhesive to facilitate removal.
 - **PET OWNERS: Do not use isopropyl alcohol over carpeted or hardwood floors!**

Remove the Device:

- Place the patient in right lateral recumbency.
- REMOVE THE RECORDER and place in the silver bag (electrostatic shielded bag).
- Grasp the top tab from the Battrode (small tab above the Recorder) and gently pull back.
- Begin wiping this section with adhesive remover in quick, even, and repeating strokes.
- Continue pulling the tab down as you wipe the adhesive remover in each section.
 - The adhesive remover tends to work within about 2-3 seconds after applying.
 - Use isopropyl alcohol or the additional adhesive removal pads as necessary.
 - Continue this process until the entire unit is detached from the patient.
- Once detached, apply the Battrode (adhesive section down) to the **white label**.
- Remove the Recorder and **place in the silver anti-static pouch** – reseal the pouch.
- Remove the clear seal sticker from the diary (or plan to use packing tape to secure the box).
- Place the **Recorder, Battrode, Diary, and any remaining stickers** into the shipping box.
 - The adhesive remover can be discarded.
- Seal the box with the clear sticker (or tape) and write return mailing address in the top left corner of the package.
- Place the box in a **US POSTAL SERVICE** box OR return to a local post office.
 - **Shipping is pre-paid and returns to us directly.**

PLEASE NOTE

**Due to the detailed analysis performed on each recording,
it may take up to 10-14 days to obtain results.
We will send a report as soon as it is available.**

Please email info@olympicvetcardio.com that the device is in the mail.
We have all tracking information saved for each device.