

# CAM Success Checklist



- Remove contents from box.
- Place patient in right lateral recumbency.

## Prepare the Patient:

- Clip left parasternal/axillary region with 1" margins around area for device.
- Apply 3 rounds of isopropyl alcohol wipes to attachment area.
- Allow area to dry completely (check with back of index finger).

## Prepare the CAM Device:

- Remove Battrode from white pouch, leaving protective plastic in place.
- Remove Recorder from silver pouch.
- Snap RECORDER into BATTRODE.
- Confirm the LED light flashes (generally within 2-3 seconds after insertion).
  - If not, remove the Recorder and reinsert. If no buzz, stop the attachment.

## Attach to Patient:

- Remove Battrode from the plastic sheet - top down (use the tabs at the top).
  - Peeling gently to ensure the adhesive does not stick to itself or the human's arm.
- Place onto patient chest with the Recorder just above the costochondral junction and work down the chest, following the angulation of the ribs (tail angled caudally).
  - It is ok if the tail wraps under the chest and to the right apical region (as long as the patient was prepared correctly for this).
- ESSENTIAL STEP:**
  - **Note the time the device is applied to the skin (recording begins).**
  - **DOCUMENT TIME in the included Patient Diary.**
- Gently rub the edges of the Battrode with your fingers to warm the adhesive (~1 min).
- Firmly press the entire unit against the chest with the palm of your hand (a few seconds).
- Gently* press the silver button on the Recorder to confirm a final LED flash.
- Allow the patient to stand.

## Protect the CAM Device:

- Apply Vetwrap over the device/chest in a cross-your-heart style bandage.
  - Use a small amount of tape or Elastikon if necessary to help secure bandage.
- Place a Thunder Shirt** (or similar padded vest) over this wrap for extra protection.
  - **Additional Fees May Apply if device is not properly protected.**
- Patient returns home for owner to monitor and maintain Patient Diary.

Length of Recording: 48 hr / 7 Day

REMOVAL DATE: \_\_\_\_\_ / TIME: \_\_\_\_\_